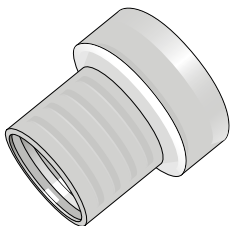
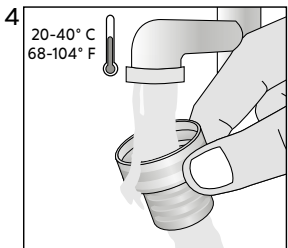
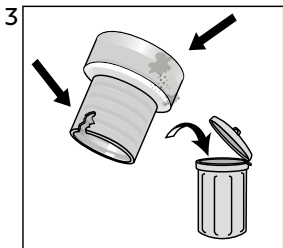
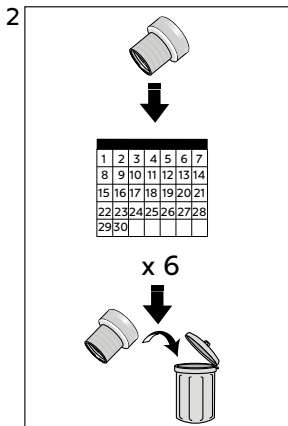
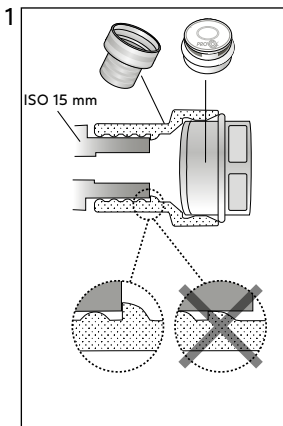
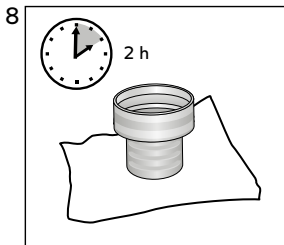
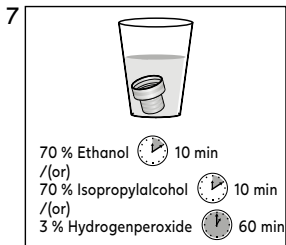
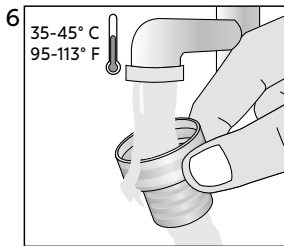
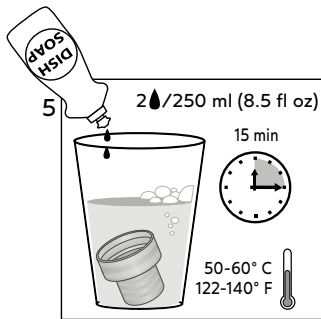
PROVOX[®]

HME Cassette Adaptor

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Contents

EN - ENGLISH.....	5
DE - DEUTSCH	5
NL - NEDERLANDS.....	6
FR - FRANÇAIS.....	6
IT - ITALIANO.....	7
ES - ESPAÑOL.....	7
PT - PORTUGUÊS	8
SV - SVENSKA.....	8
DA - DANSK	9
NO - NORSK.....	9
FI - SUOMI	10
IS - ÍSLENSKA	10
PL - POLSKI	11
HR - HRVATSKI	11
SR - SRPSKI.....	12
EL - ΕΛΛΗΝΙΚΑ	12
TR - TÜRKÇE.....	13
HY - ՀԱՅԵՐԵՆ	13
RU - РУССКИЙ	14
MS - BAHASA MELAYU.....	14
JA - 日本語.....	15
ZHTW - 繁體中文.....	15
HE - עברית	16
AR - العربية.....	16

EN-ENGLISH

Intended Use

The Provox HME Cassette Adaptor is intended to facilitate a connection between Provox HME Cassettes and on the market available tracheal cannulas with ISO-cone (15mm).

WARNINGS

- DO NOT use HME Cassette Adaptor on a single lumen tube (tube without an inner tube) unless the patient or caregiver is able to reinsert the tube themselves if they have to remove it in case of emergency.
- Single patient use only! Reuse between patients may cause cross contamination.

Cleaning and disinfection

- Clean and disinfect Provox HME Cassette Adaptor before each use. Only use drinking water to clean and rinse the device (Fig. 4).
- Place Provox HME Cassette Adaptor in water-based mild soap for 15 minutes (Fig.5).
- Rinse Provox HME Cassette Adaptor with water (Fig. 6).
- Disinfect Provox HME Cassette Adaptor at least once a day (Fig. 7) with one of the following methods: Ethanol 70% for 10 minutes or Isopropylalcohol 70% for 10 minutes or Hydrogenperoxide 3% for 60 minutes.
- Let Provox HME Cassette Adaptor dry at least for 2 hours or until it looks dry (Fig. 8).

Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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