
PROVOX[®] Luna[®]

MD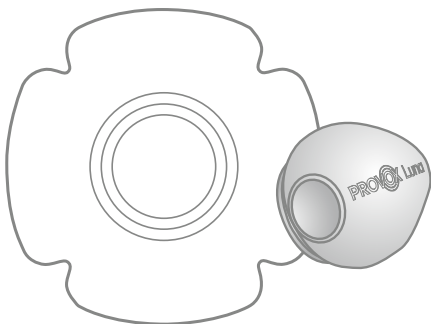

CE

42 °C
108 °F
RT
2 °C
36 °F


Provox Luna HME

**UK
CA**


30 °C
86 °F
RT
2 °C
36 °F


Provox Luna Adhesive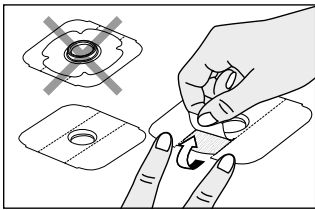


1.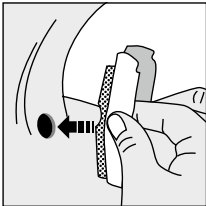


2.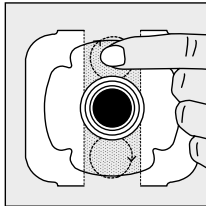


3.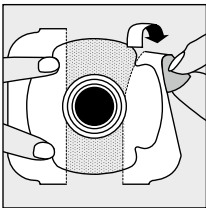


4.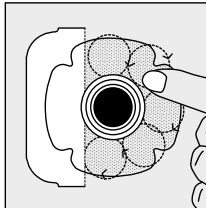


5.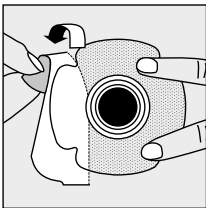


6.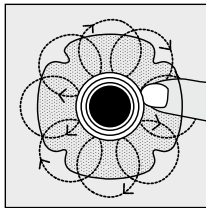


7.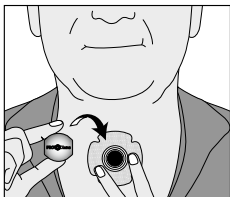

8.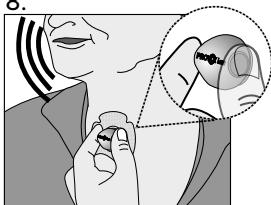

9.

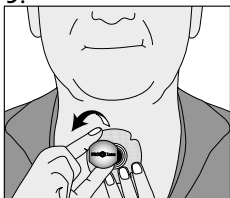10.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles ; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ühilduvad tooted; Savietojami izstrādājumi; Suderinami gaminiai; Kompatibilní produkty; Kompatibilis termék; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Produse compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Совместимые изделия; Uyumlu Ürünler; Համատեղելի արտադրանքներ; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; 相容产品

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1. Descriptive Information

1.1 Intended use

Provox Luna HME is a single use heat and moisture exchanger, attachable to the Provox Luna Adhesive, for night-time use after total laryngectomy.

Provox Luna Adhesive is a skin-friendly, single use adhesive that provides attachment for Provox Luna HME for night-time use after total laryngectomy.

1.2 CONTRAINDICATIONS

The devices should only be used in accordance with the Instructions for Use. Patients without the physical, cognitive, or mental ability required to attach, remove or operate the devices themselves, should not use the devices independently and should only use them if they are under sufficient supervision of a clinician or a trained caregiver.

Only valid for HMEs

The devices should not be used by patients with a low tidal volume, as the added dead space may cause CO₂ (Carbon dioxide) retention, see 1.4 Technical data.

1.3 Description of the device

Provox Luna HME is a soft heat and moisture exchanger that helps to keep the lungs healthy by humidifying the inhaled air.

Provox Luna Adhesive consists of an adhesive base, a peel-off liner and a soft connector for Provox Luna HME. Provox Luna Adhesive base is a skin-friendly hydrogel adhesive intended for night-time comfort and skin rest.

Caution: Only original components should be used as other accessories may cause product malfunction.

Information

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

1.4 Technical data

Dead Space	7 ml
Pressure drop* after 30 min at	
-10 l/min	0.10 hPa
-30 l/min	0.55 hPa
-60 l/min	1.90 hPa
Moisture loss* at	
-VT= 500 ml	17.7 mg/l

*According to ISO 9360

1.5 WARNINGS

- Do not use Provox Luna HME or Provox Luna ShowerAid with any other adhesives or other accessories unless indicated in this Instructions for Use.
- Do not use Provox Luna Adhesive with any other HME or accessories unless indicated in this Instructions for Use.
- Provox Luna Adhesive should not be worn during radiotherapy treatment sessions as it will alter the delivery of radiotherapy to that area. As Provox Luna Adhesive is a hydrogel, it is possible to use this adhesive during the night between radiotherapy sessions; however, clinicians should be aware that this may not be appropriate for all patients and use of Provox Luna Adhesives needs to be assessed both on an individual basis and reviewed regularly through the course of radiotherapy treatment.

1.6 PRECAUTIONS

- The hydrogel adhesive material absorbs moisture from the skin and the environment (humid air, sweat etc.) and may come loose if it becomes too wet. Consult your clinician if you need a stronger adhesion.
- Be careful not to exert pressure on Provox Luna HME unintentionally. Pressure may reduce the size of the side openings and cause difficulty breathing.
- Do not reuse Provox Luna HME as the risk of potential infections may increase due to bacterial colonization of the HME.
- Do not rinse Provox Luna HME with water or any other substance since this will substantially reduce the function of the HME.
- Do not remove the HME foam, as the HME function will be lost.
- Provox Luna Adhesive may be used on irritated skin. However, if skin irritation persists for more than 14 days, stop using the adhesive and consult your clinician.

- If new skin irritation develops, stop using Provox Luna Adhesive and consult with your clinician.
- Do not administer medicated nebulizer treatment over the device since the medication can become deposited in the device.
- Breathing may be restricted if humidifiers or heated humidified oxygen via a mask over the tracheostoma is used, the HME can become too wet.

2. Instructions for use

2.1 Operating Instructions

Clean the skin around the stoma with a Provox Cleaning Towel and/or soap and water to remove any oil and adhesive residues.

Let your skin dry before attaching Provox Luna Adhesive.

Caution: When cleaning the skin from residual glue, prevent particles/fluids from entering the tracheostoma.

Caution: Do not use skin barrier products or products (e.g. soap, skin cream) that leave oily residue on the skin that may prevent the adhesive from sticking properly.

For instructions on how to apply the products, please see fig. 1-8.

In order to get a seal for speech, occlude Provox Luna HME by covering the openings on the sides (fig. 9).

Provox Luna HME can be removed (as in fig. 10) if mucus needs to be cleaned from the stoma. Provox Luna Adhesive can stay in place during cleaning of the stoma and changing the HME.

In the morning, carefully remove Provox Luna Adhesive. Using a little bit of water at the side and underneath the adhesive helps to remove the adhesive.

Caution: When taking a shower with Provox Luna Adhesive use Provox Luna ShowerAid and Provox Adhesive Strip. The hydrogel adhesive material may absorb water and detach.

2.2 Device lifetime

Provox Luna HME and Provox Luna Adhesive are for single night time use and should be replaced in the morning with another Provox HME and Adhesive, or more often if needed.

2.3 Accessories

Use Provox Luna ShowerAid together with Provox Luna Adhesive and Provox Adhesive Strips when taking a shower.

Provox Adhesive Strip is a single use device intended to protect Provox Luna Adhesive while taking a shower. Please see Instructions for Use for Provox Luna ShowerAid.

2.4 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

3. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical UK Ltd Tottle Road
Atos Medical AB